



PATIENT

# MedVac Immobilizer

## INF-VMR33PUBB, INF-VMR38X01



VMR33PUBB - Pediatric (Newborn to 1 year old)



VMR38X01 - Child (1 year to 4 years old)



INFAB's Med-Vac Immobilizer is a reliable solution designed to securely fasten around pediatric and child patients reducing the amount of motion during MRI, CT or X-Ray imaging and decreasing the number of repeat scans. Using our pediatric immobilization device eliminates the need for sedation during the most sensitive stages of pediatric development.

The air-tight vacuum bag is filled with evenly distributed beads which helps secure the infant without exerting pressure. The chamber system allows for quick and easy application and a quality image.

Benefits: The chamber system facilitates swift and effortless application. Certified "MRI Safe" according to FDA Consensus Standard ASTM F-2503-13.

**Directions For Use:** wrap the vacuum bag around the patient, secure the straps, connect to a pump, and evacuate the air. The bag molds gently around the newborn, ensuring immobilization without discomfort. Upon completion, release the valve to allow the bag to refill with air, then unfasten the straps. In-room vacuum pumps are available, or a manual pump can be purchased separately.

- Effectively reduces the amount of motion during MRI scans, decreasing the need for repeat scans
- Eliminates the need for sedation and provides comfort for patients
- Chamber system allows for quick and easy application
- MRI Safe as defined by FDA Consensus Standard ASTM F-2503-13
- Ensures high-quality imaging results

<b>Model</b>	INF-VMR33PUBB - Pediatric (Newborn to 1 year old) INF-VMR38X01 - Child (1 year to 4 years old)
--------------	---

<b>Description</b>	MedVac Immobilizer
--------------------	--------------------

<b>Modalities</b>	Best Seller, MRI
-------------------	------------------

<b>Substrate</b>	Heavy Duty Vinyl
------------------	------------------

<b>Sterile</b>	N
----------------	---

<b>Latex Free</b>	Y
-------------------	---

<b>Single Use</b>	N
-------------------	---



REV002

1040 Avenida Acaso  
Camarillo, CA 93012, USA

tel : 805.987.5255  
fax: 805.482.8424

info@infabcorp.com  
www.infabcorp.com

